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ENGLISH

1 General information

This raising harness for horses and cattle has been manufactured and controlled with a great deal of care and under very rigorous quality criteria. So the requirements for safe use have been observed. Now it is up to you to use this harness in the CORRECT way! **THEREFORE READ THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE USING FOR THE FIRST TIME!** Please keep these instructions for use with the product, so you will be able to refer to them in case of problems and fill in the TEST SHEET on the last page carefully. In case repairs are necessary or complaint it is absolutely essential to send us this test sheet together with the product.

2 Easy and straightforward putting on

Suitable for horses and cattle. (note the stamp on the front and back outer or inner side!) The middle section (main supporting part) can be put on easily, without fittings with large loops. (Therefore it can easily be pulled through under the animal when it is in a lying position.) With this middle part the animal can even be raised in an emergency, at least helped up (the main part to be fastened with the green belt and insertion buckle over the shoulder. See fig. A). In case of rescuing of a horse it is recommended to fix the head with a halter, which is attached to the anchor point.

3 Protection against slipping off

The protection against slipping off at the front (orange belts) and the back (orange belts with padding and belt with elastic band) should be attached

immediately or after having raised the animal, while the back protection with blue padding is designed in a completely removable way. The assembly and tightening of the belts is carried out with simple double-frame buckles of light metal (size adjustment and support distribution for balance). (See fig. B). Tighten (balance) only after having raised the middle part a bit! (take care of numbering!) The udders of cows are protected by avoiding belts in this area. The back slipping protection runs outside over the thighs of the hind leg.

The animal should be absolutely calmed down by a vet before putting on the harness or raising, in order to avoid injury to man and animal. The vet should remain during the rescue procedure!!!

4 Use

After each use carry out a visual inspection of the raising harness. While using take care that the harness is not damaged by mechanic influence (sharp edges etc.) nor comes in contact with harmful chemical substances.

5 Cleaning

The harness should be cleaned if necessary, best dry and with soft brushes. In case of intensive soiling warm water and mild detergents may be used. Let it dry in the air (not in the sun, or by direct heat or tumble dry!). Do not use chemical detergents or high pressure cleaners.

6 Repairs

Eventual repairs may only be carried out by the manufacturer. For safety reasons however, repairs should be avoided as far as possible.

7 Storage

The harness must be stored in a dry place protected from light. It must not be stored near heaters or come into contact with aggressive substances (e.g. oil, chemical substances). In case of doubt we recommend asking the manufacturer.

8 Period of use

The raising harness for horses and cattle has to be discarded:

- In case of damage to supporting parts, which are essential for safety
- After expiry of the recommended period of use of max. 12 years

Periodic inspections should be carried out within the recommended period of use.

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We recommend that personal protective equipment is always used by one person only! This personal protective equipment is used by name(s):
Date of purchase: bought at:
Date of 1st use:
Inventory no.:
Periodic inspections have to be carried out by a COMPETENT PERSON at least once a year!
Notes:

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